

Coolsense™ Pain Numbing Applicator

Balance Medical Pty Ltd



Coolsense™ Pain Numbing Applicator

- * New, simple to use device
- * Numbs the site within 10 seconds
- * TGA approved



Coolsense™ Product Overview

- * Keep in freezer to maintain optimum operating temperature of -2 to -4 degrees Celsius
- * 70% Isopropyl alcohol protects skin from risk of cold burn
- * Alcohol-filled cartridge also provides continuous disinfection mechanism when in contact with the device
- * Device numbs site immediately (within 10 seconds)



Used by over 40,000 doctors
worldwide



Coolsense^{TMTM} Product Summary

- * Numbs the site prior to any type of injection/ cannulation procedure
- * Use in place of dermal anaesthetic creams such as EMLA or ANGEL cream
- * Reusable device
- * No batteries or electricity required
- * Highly cost effective
- * TGA approved as a Class 1 non-sterile device



Coolsense™ Applications

- * Aesthetic procedures
- * Venipuncture
- * Cannulations
- * Paediatrics
- * Emergency department
- * Operating Theatres
- * Dialysis
- * Oncology
- * Diabetes
- * Needle-phobic patients



Coolsense™ – How it works

- * Store device continuously in freezer
- * Coolsense will reach -2 to -4 degrees Celsius
- * Remove from freezer and apply to skin for up to 10 seconds
- * Administer injection immediately
- * Wipe device with an alcohol wipe before replacing cartridge and returning to freezer



Benefits of Using Coolsense™

- * Apply device only to site to be injected rather than multiple sites (as with anaesthetic creams)
- * No waiting for dermal anaesthetic cream to work
- * No issues with vasoconstriction or reactions
- * Reduces initial pain experienced by patient making for a more comfortable experience



Thank you

Coolsense™ – makes sense to lose that sense of pain

